AED Presentation FM Safety Committee





Staff Development Day October 9th, 2018

Introduction



- This collection of material was presented on Octob et a skills for its 9th, 2018, at Staff Development Day, as an initiative of the FM Leadership Team and the Safety Committee.
- The information assembled in this PowerPoint
 Presentation was been developed as an overview of the AED, in collaboration with EMS Safety.
- The intent of this presentation is to familiarize our staff and faculty with the location and use of the AED devices strategically deployed at FM.

CPR/AED

AED Overview



What is an AED?

— Automated External Defibrillator (AED) Device is a small, lightweight device that analyses a person's heart rhythm and can recognize irregular heart rhythms such as ventricular fibrillation (VF) or ventricular tachycardia (VT), also known as Sudden Cardiac Arrest (SCA).

-What is SCA?


• SCA is a condition in which the heart suddenly and unexpectedly stops. An AED is a portable device that checks the heart rhythm and can, when necessary, send an electric shock to the heart to try to restore a normal rhythm.



AED Ovei

- **Lea**rn w
- Understand
 early ac
- Know th AED.



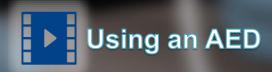

AED Overview





Where are AEDs?

- Public buildings: airport, government office
- Meeting places: stadium, mall
- Difficult or remote locations
 - High rise
 - Construction
 - Commercial airliner
- Work/Office/Gym

What are AEDs?

- Reads heart rhythm, shocks heart
- Like a computer reboot
- Most effective in first 3 minutes after SCA
- Goal: restore normal heartbeat




- not shock if someone who doesn't need it!
- Know where the nearest AED is located!

AED Overview 5

Where are the AED locations at FM?






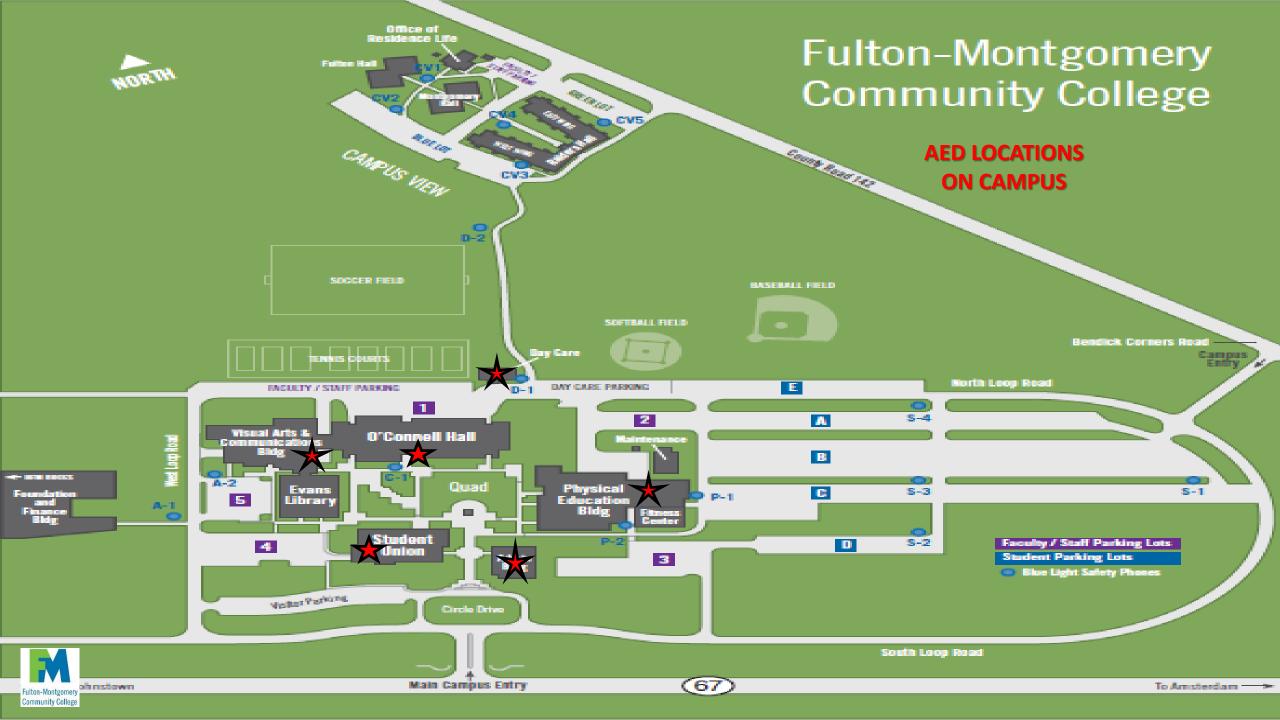




Student Union Building



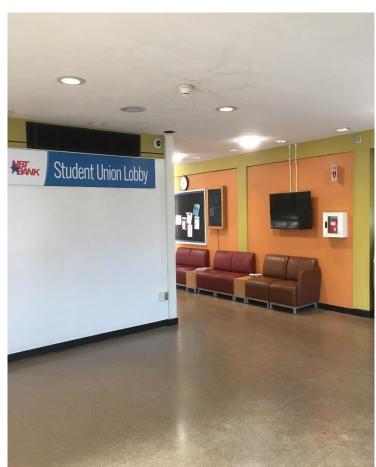

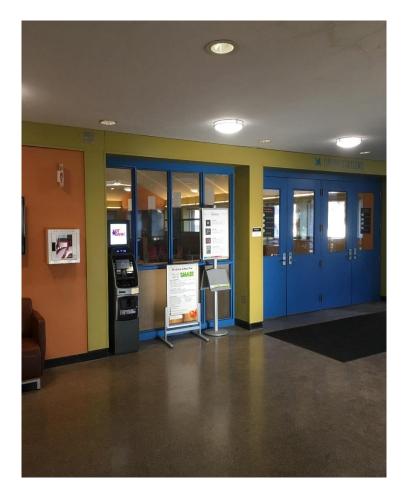








AED - #2

Visual & Performing Arts Building (AKA The "Annex")





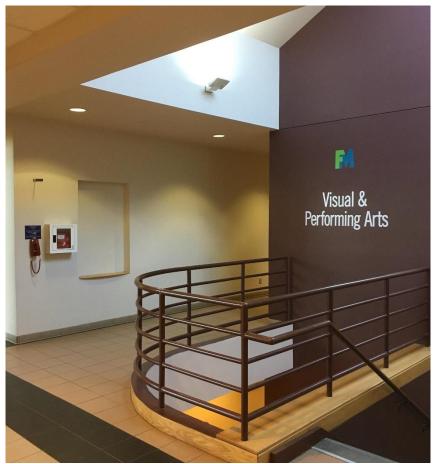









O'Connell Hall Main Floor at Central Staircase



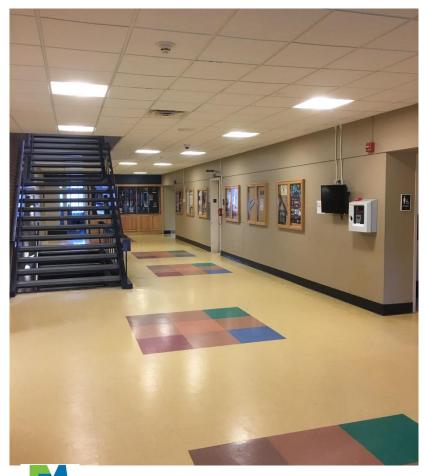

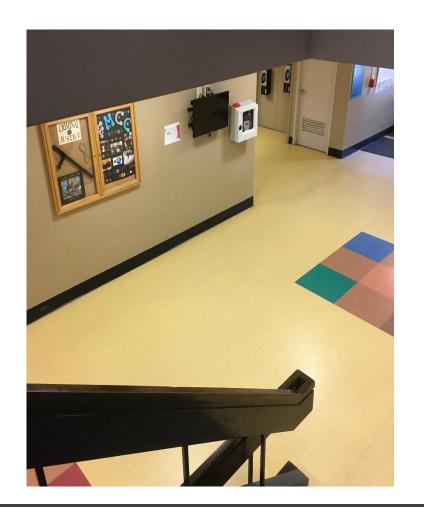








Physical Education Building



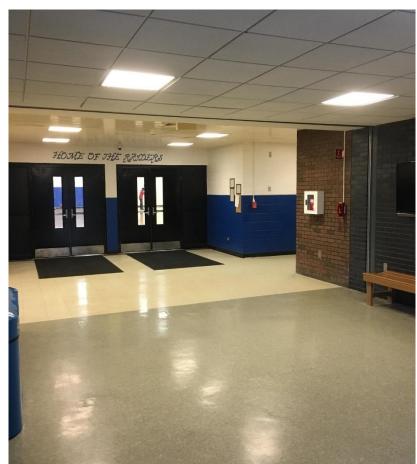







Daycare Building







Daycare Building











Student Welcome Center



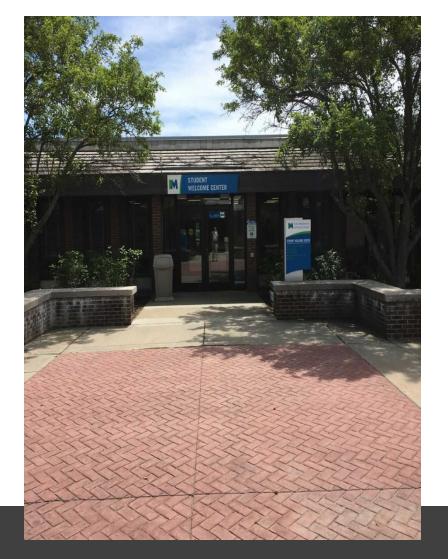


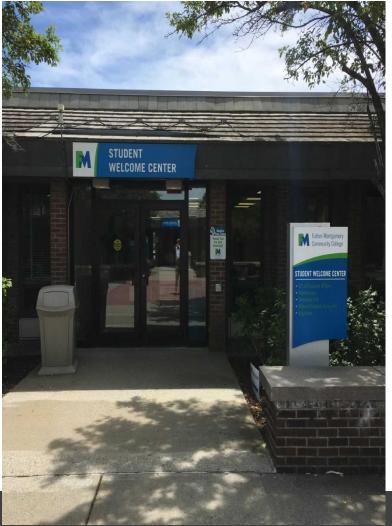



Student Welcome Center



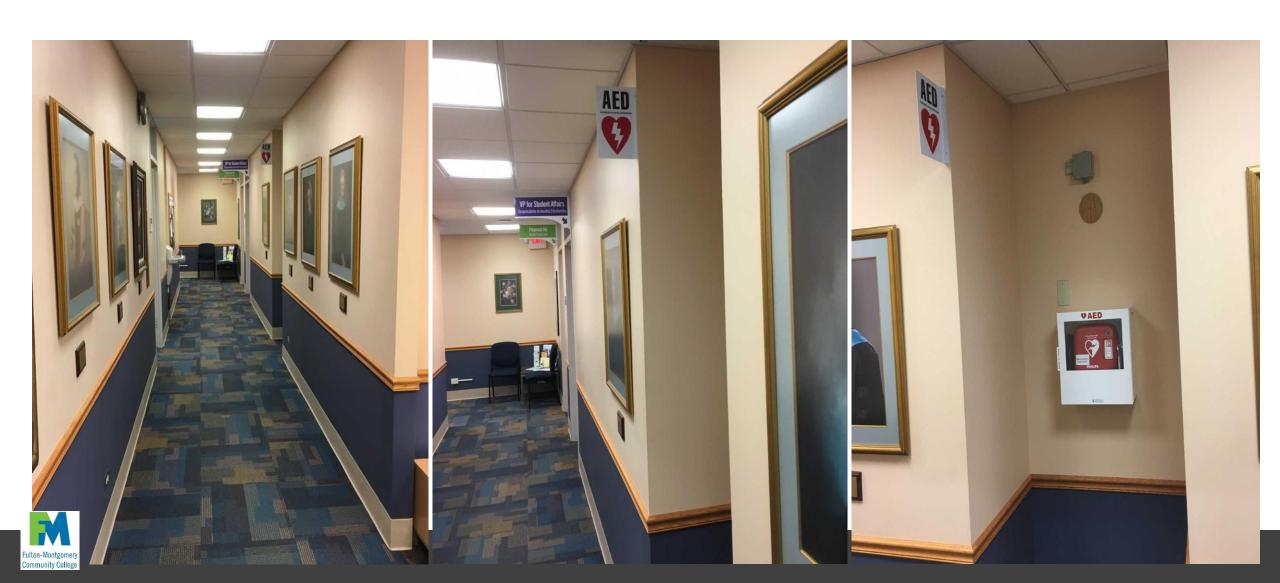

- Learn how to use an AED.
- Understand the safety precautions when using an AED.
- Skill: Using an AED

Turn on the AED & Follow Prompts







Know your AED:

- Where is it?
- How do you turn it on?
- What accessories are stored with it?

- Use an AED as soon as it is available
- Place the AED near the person's head
- Do not remove it from the case
- Turn on the AED
- Follow the prompts

Apply AED Pads



- Apply AED pads to bare skin
 - Move or cut clothing before applying pads
 - If the person is wearing a bra, cut it off





- Peel and place 1 pad at a time
- Follow the pictures on the pads
- Right upper chest, just below collarbone
- Lower left side of the chest

Clear Victim and Shock



After pads are applied, AED prompts you to stop CPR

- Do not touch the person while AED analyzes heart rhythm
- If no shock advised, AED will prompt you to continue CPR

When a shock is advised:



Clear victim

- Loudly state, "Clear!"
- Look up and down the person
- Ensure no one is touching person or clothing

Press shock button

- Some AEDs shock automatically (no button to push)
- Resume compressions when AED prompts

Follow AED Safety Precautions







How do you make this scene safe for AED use?

- Remove the person entirely from the water.
- Quickly dry the chest.
- If the victim is lying in a puddle, move to a dryer location.



Oxygen

- May be used by the victim or a rescuer
- Turn off oxygen or move it several feet away

Standing Water

- Do not use the AED in a puddle
- Move to a dryer area before using the AED
- Rain or small amounts of water will not interfere
- Quickly dry the chest of water or excess moisture before attaching pads

What is wrong in this picture?






When a Shock is Advised:

Clear victim

- Loudly state, "Clear!"
- Look up and down the person
- Ensure no one is touching person or clothing

Then Press shock button

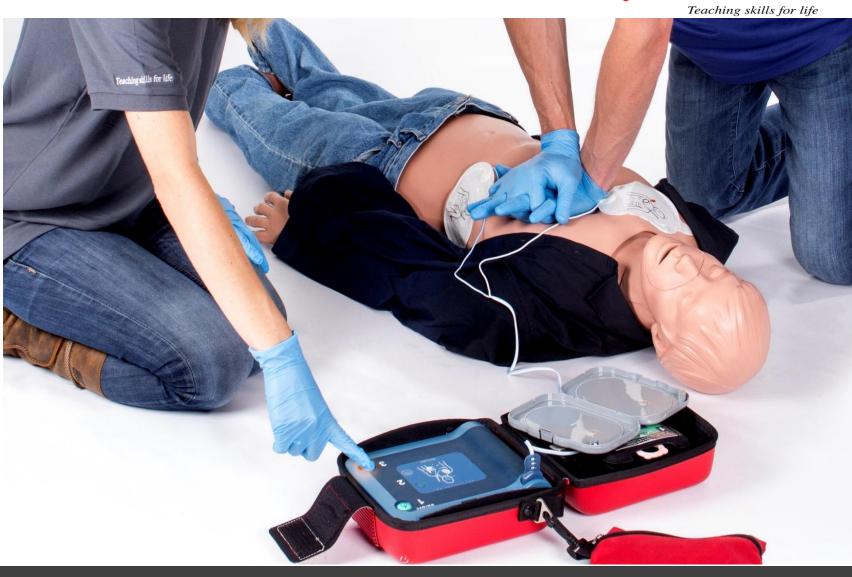



AED Use





- Place AED near person's head
- Turn on AED and follow prompts



Apply pads



- "Clear!" victim before shocking
- Press shock button




 Start CPR when prompted by the AED

AED Considerations

- Identify actions to take when something interferes with AED pad placement.
- Understand the importance of keeping an AED ready to use.

AED Pad Placement Considerations



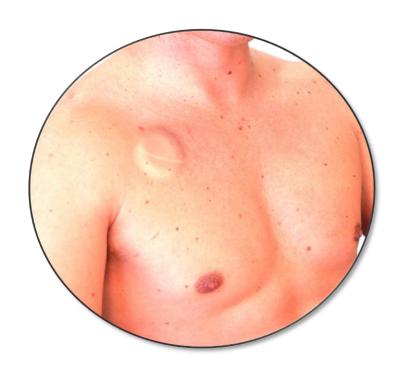

Implanted device appears as a lump under skin

Implanted Device

- Do not place pad directly over implanted device
- Move pad at least 1 inch from device

Medication Patch

 If in the way, remove patch with gloved hand and wipe skin clean

Chest Hair

- If excessive, quickly shave chest in area of pad placement
- If have extra pads, apply firmly, quickly remove, and apply new pads

AED Considerations 28

Maintenance & Troubleshooting







Store an AED ready to use

- Close to trained rescuers
- Visible sign; easy access
- With accessories
- Perform regular inspections
 - Status indicator 'Ready'
 - Pads & battery not expired
- If the AED detects a problem:
 - Follow AED prompts
 - FM Public Safety checks our AEDs.



If you can't quickly troubleshoot a problem, continue CPR and have another responder troubleshoot the AED.

AED Considerations 29

New York Good Samaritan Law



New York State Public Health Law Section 3000-a

- § 3000-a. Emergency medical treatment.
- 1. Except as provided in subdivision six of section six thousand six hundred eleven, subdivision two of section six thousand five hundred twenty-seven, subdivision one of section six thousand nine hundred nine and sections six thousand five hundred forty-seven and six thousand seven hundred thirty-seven of the education law, any person who voluntarily and without expectation of monetary compensation renders first aid or emergency treatment at the scene of an accident or other emergency outside a hospital, doctor's office or any other place having proper and necessary medical equipment, to a person who is unconscious, ill, or injured, shall not be liable for damages for injuries alleged to have been sustained by such person or for damages for the death of such person alleged to have occurred by reason of an act or omission in the rendering of such emergency treatment unless it is established that such injuries were or such death was caused by gross negligence on the part of such person. Nothing in this section shall be deemed or construed to relieve a licensed physician, dentist, nurse, physical therapist or registered physician's assistant from liability for damages for injuries or death caused by an act or omission on the part of such person while rendering professional services in the normal and ordinary course of his or her practice.

Gross Negligence.

Under New York law, misconduct that rises to the level of gross negligence must show "reckless indifference to the rights of others." The conduct must show a "failure to use even slight care or conduct that is so careless as to show complete disregard for the rights and safety of others."

CPR/AED